

USAGE

1. FACE THE MASK WITHOUT THE NOSE CLIP, AND THE NOSE CLIP SHOULD BE ON TOP;
2. HOLD THE MASK ON YOUR FACE AND AGAINST YOUR CHIN;
3. PULL THE STRAPS ON BOTH SIDES AND HANG ON THE EARS, AND ADJUST TILL YOU FEEL COMFORTABLE;
4. PUT YOUR FINGERS IN THE MIDDLE OF THE METAL NOSE CLIP, AND MOVE YOUR FINGERTIPS TO BOTH SIDES UNTIL THE NOSE CLIP IS FULLY PRESSED INTO THE SHAPE OF THE NOSE SLOPE;
5. PUT YOUR HANDS ON YOUR FACE TO CHECK THE TIGHTNESS WITH YOUR FACE.



PRODUCT INFORMATION

PRODUCT NAME: PROTECTIVE MASK

PRODUCT FEATURE AND COMPOSITION: THIS PRODUCT IS LAMINATED FROM MULTIPLE LAYERS OF THIN FABRIC

SIZE: ADULT

IMPLEMENTATION STANDARD: GB 2626-2006

DATE OF MANUFACTURING: SEE INNER BOX CERTIFICATE

VALIDITY: 2 YEARS

PRODUCTION DATE: 2020.04

PACKING SPECIFICATION: 10 PCS/CASE

STORAGE: STORED IN A VENTILATED, DRY, HYGIENIC AND NON-CORROSIVE GAS PLACE.

CAUTION: PATIENTS WITH DIFFICULTY BREATHING OR CONGENITAL HEART DISEASE, PLEASE USE WITH CAUTION;

THIS PRODUCT CANNOT BE USED AS A GAS MASK IN A TOXIC ENVIRONMENT.


MASK

MASK

Non Medical Mask

Advanced environmental protection non-woven fabric, multi layer fabric can effectively resist fine particles and droplets in the air

KN95 PROTECTIVE MASK
Ear mask

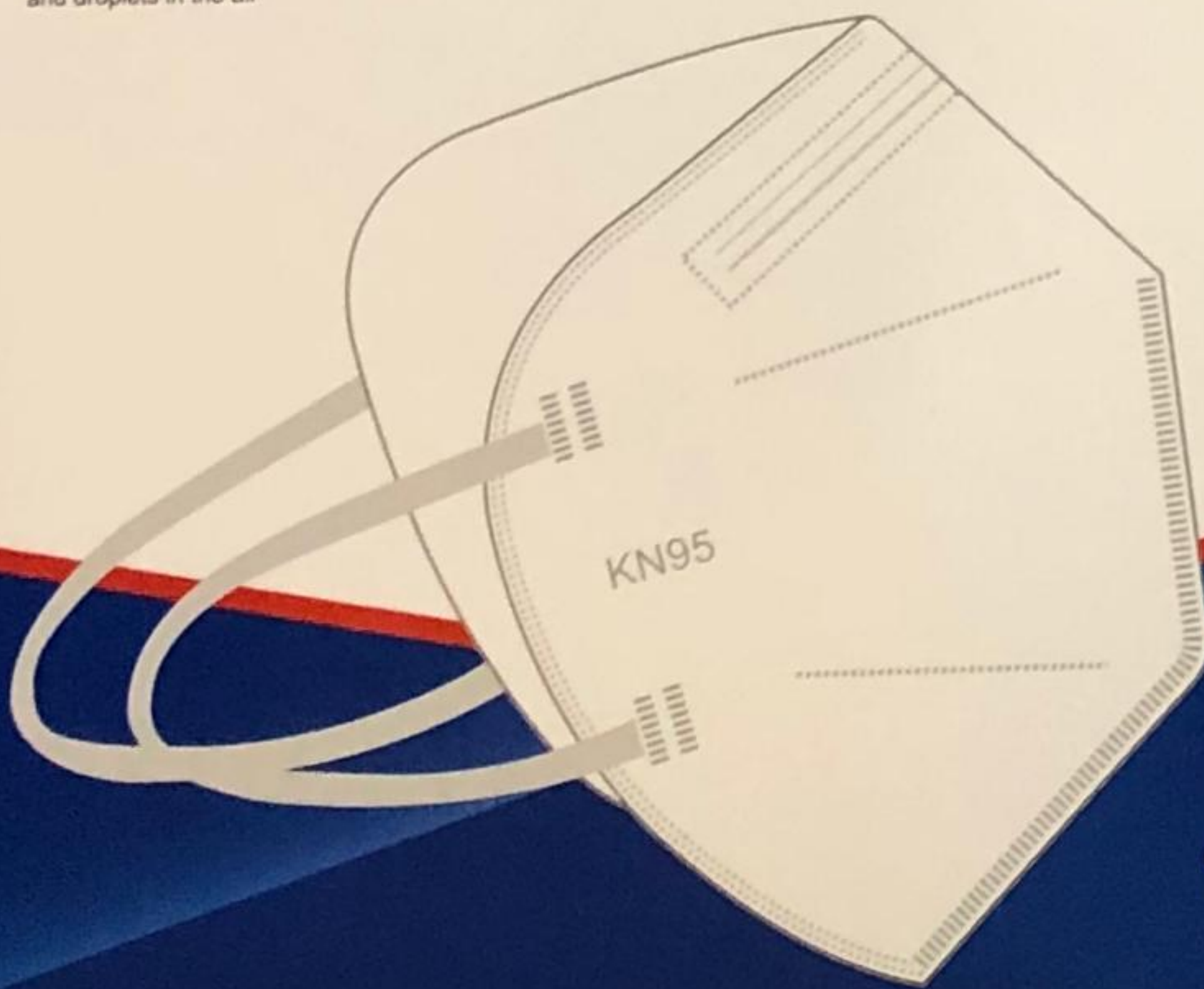

10
pcs

MASK

★★★★★
KN95

MASK

MASK

Folding Protective Face Mask



Non-medical

10
pcs

USAGE

Mask on mouth and nose, prevent inhalation of dust.

Wearing method

- 【1】 Face the mask without nose clip place the nose clip over the mask, noid the mask on your face, put the mask on your chin.
- 【2】 Just pull the earband to the outside of both ears and harit.
- 【3】 Put your fingers in the middle of the metal nose clip, press in while moving the ingertips to both sides along the nose clip. until the nose clip i fully pressed into the shape of the bridge of the nose.
- 【4】 Put your hands on your face to check for face tightness.



Product information

【PRODUCT NAME】 EAR FACE MASK

【PRODUCT PERFORMANCE STRUCTURE AND COMPOSITION】

This product is made of multilayer thin fabric

【PRODUCTION DATE】 See the outer package for batch number

【SHELF LIFE】 3 years

【NOTE】 Patients with dyspnea or congenital heart should use it carefully
This product can not be used as gas muask in toxic environment



Certificate's
Holder:

Shantou Himiss Clothing Co., Ltd
NO.737 of old Hehui Road, Heping Town,
Chaoyang Distriet, China

Certification ECM
Mark:



Product:
Model(s):

Non medical mask
KN95

Verification to:

Standard:
EN 149:2001+A1:2009

related to CE Directive(s):
R 2016/425 (Personal Protective Equipment)

Additional information and clarification about the Marking:



The manufacturer is responsible for the CE Marking process. This document has been issued on the basis of the regulation on ECM Voluntary Mark for the certification of products. RG01_ECM rev.3 available at: www.entecerma.it

Issuance date: 20 March 2020

Expiry date: 19 March 2025

Reviewer
Technical expert
Amanda Payne

Approver
ECM Service Director
Luca Bedonni



Via Ca' Bella, 243 – Loc. Castello di Serravalle – 40053 Valsamoggia (BO) - ITALY
 ☎ +39 051 6705141 📠 +39 051 6705156 ✉ info@entecerma.it 🌐 www.entecerma.it